EXPASYL™

Class I medical device / Dispositif médical de classe I – CE

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Read the instructions in the notice carefully – All the compulsory information requested for a good use of this device is in the Product Features Summary available on the laboratory's website – Not reimbursed by health insurance organisations. March 2016.

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Tel + 33 (0) 556 34 06 07 • Fax + 33 (0) 556 34 92 92

E-mail : pierre-rolland@acteongroup.com • www.acteongroup.com

The ACTEON logo, featuring the word "ACTEON" in a bold, sans-serif font, with a blue swoosh underline that starts under the 'A' and ends under the 'N'.

EXPASYL™

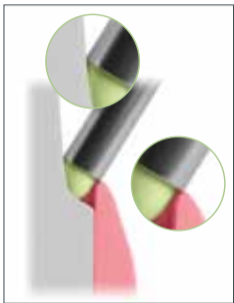
The ACTEON logo, featuring the word "ACTEON" in a bold, sans-serif font, with a blue swoosh underline that starts under the 'A' and ends under the 'N'.

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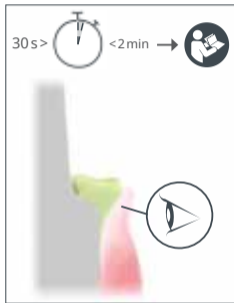
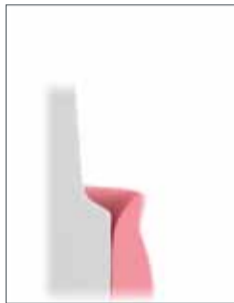
Medical device category I

1. Rinse and dry the preparation lightly. 2. Place the extremity of the cannula so as to obtain a closed space. Slowly extrude the material. 3. At the end of the application press the extremity of the cannula on the tooth and turn it. The string of paste easily detaches. 4. Leave Expasyl to work for 1 to 2 minutes. The whitening of the gums shows the compression by the material. 5. Eliminate with a gentle air-water spray. 6. The sulcus is open, the site is perfectly dry. Without lesion or pain. Bleeding does not start again.

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MZ 3549 a - Copyright © 2016 PIERRE ROLLAND - Tous droits réservés.