


HOW TO USE ACTEON® ULTRASONIC DEVICE TO LIMIT AEROSOL CONTAMINATION

CLINICAL PROTOCOL



LET'S FIGHT
CORONAVIRUS
TOGETHER

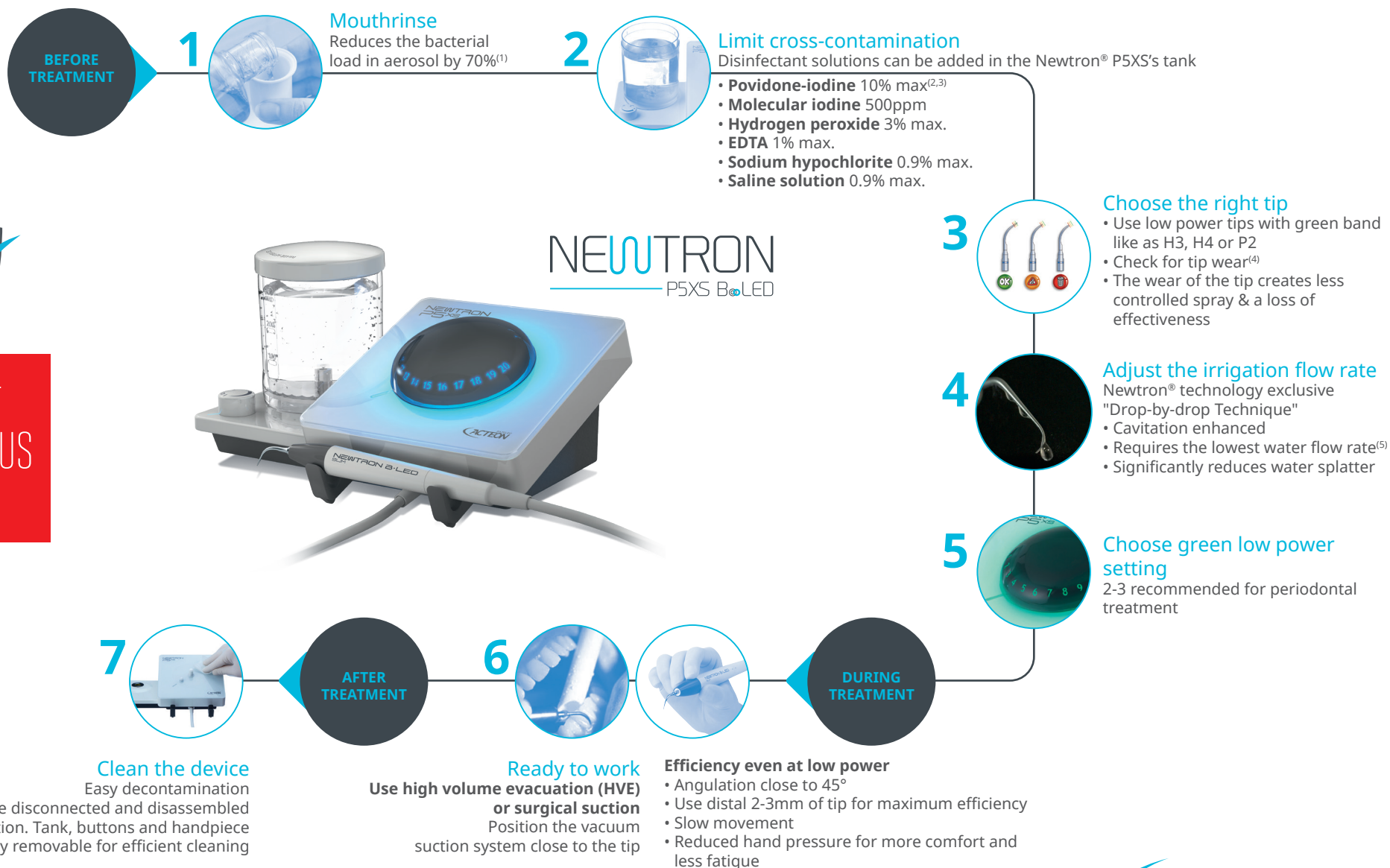

(1) Narayana TV, Mohanty L, Sreenath G, Vidhyadhari P (2016) Role of preprocedural rinse and high volume evacuator in reducing bacterial contamination in bioaerosols. J Oral Maxillofac Pathol 20(1):59-65. doi: 10.4103/0973-029X.180931 (2) In vitro bactericidal and virucidal efficacy of povidone-iodine gargle/mouthwash against respiratory and oral tract pathogens (3) Slight risk of discolouration of plastics; rinsing between each patient is recommended (4) It is recommended to change tips after 80 uses - Webinar with Dr Gagnot - 21/05/2020 "How to use Acteon ultrasonics during Covid-19?" (5) U.Van der Velden, TJG.Koster, AJ.Feilzer, MF.Timmerman, GA.Van der Weijden. Evaluation in vitro des changements de température dans les canaux radiculaires induits par un détartrage aux ultrasons. International Journal of Dental Hygiene 2014

